



Smart Glide – application in wheelchair

This instruction shows - step by step - how to use Smart Glide for placing a sling in a wheelchair.

You need two pieces of Smart Glide for this application. The Smart Glide pieces must be larger than the backrest of the wheelchair

For placing the leg straps of the sling you will need one small piece of Smart Glide covering the thigh of the user.



1 Take two pieces of Smart Glide.



2 Place the two sheets between the user's back and the backrest of the wheelchair and push both sheets behind the user.



3 Place the sling between the two sheets and push the sling down behind the user.



4 Place the leg straps and remove the Smart Glide sheets between the user and the sling.



5 Push one piece of Smart Glide under the user's thigh. Place the leg strap between the sheet and the wheelchair.



6 Pull the leg strap into position. Once the leg strap is positioned correctly the Smart Glide can be removed.