



Smart Glide – application in bed

This instruction shows - step by step - how to use Smart Glide for placing a sling under the user lying in bed.

You will need two pieces of Smart Glide for this application. The size of each piece of Smart Glide must cover from the top of the user's head and down to - and including the hip - approx. size: 100 x 140-200 cm.



1 Take two pieces of Smart Glide and fold them (Sandwich method).



2 Pull the folded sheets down to the shoulders.



3 Unfold the the sliding sheet one fold at a time. Hold the sheets while unfolding.



4 Place the sling between the two pieces of Smart Glide.



5 Pull the sling down into place under the user.



6 Once the sling is positioned correctly remove the Smart Glide between the user and the sling before lifting the user with the hoist.