

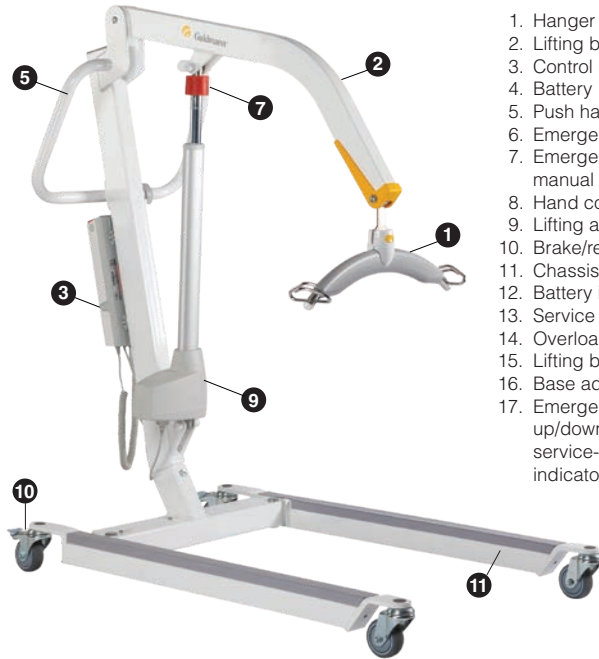


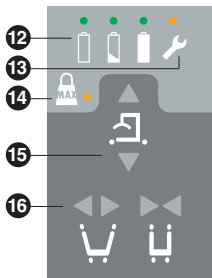

Quick Guide - GL5.2 Mobile Lifter

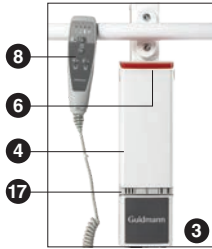

1. Hanger
2. Lifting boom
3. Control box
4. Battery
5. Push handle
6. Emergency stop
7. Emergency lowering manual (option)
8. Hand control
9. Lifting actuator
10. Brake/release
11. Chassis leg
12. Battery indicator
13. Service indicator
14. Overload indicator
15. Lifting boom up/down
16. Base adjustment
17. Emergency function: up/down, battery-, service- and overload indicator

Operators check list before use

- Make sure the lifter is moving freely on its castors.
- Operate the brakes (10) to make sure they are working.
- Lift & lower the lifting boom (2) via the control buttons (15) on the hand control (8).
- Check that the base adjustment (16) is opening and closing, via the hand control (8).
- Check the emergency function up/down by pressing the buttons (17) on the control box (3). Check the manual function by turning the knob (7) in the direction of the arrow.
- To reset the emergency stop, take off battery by pushing buttons on battery sides, and pull

battery out and push in place again. See pictures on next page.

- Make sure the hanger bar is free to rotate and swing (1), and check that the hanger bar is firmly attached to the boom (2).

Charging options

- Charger integrated in control box - plug net cable into power socket.
- When the power is plugged in, the green LED light on the control box lights up.
- The green diode on the control box (3) illuminates during charging. When fully charged 3 green diodes illuminate.



Push the grey buttons.



Pull straight out.



Push in place again.

Indicator lamps and audio warning

Status	Indicator lamps	Audio signals	Possible GL5.2 signals			
			Up	Down	Leg spread	Emergency down
Hand control						
Battery charged 75-100%	3 Green lights		x	x	x	x
Battery charged 50-75%	2 Green lights		x	x	x	x
Battery charged <50%	1 Green light		x	x	x	x
Critical low battery, two cycles left	1 Yellow light	Beeps at button activation	x	x	x	x
Lifter service	Yellow. Is lit during use and 2 minutes after		x	x	x	x
Overload	Yellow. Flashes 3 times + lights up for 10 seconds			x		x
Control box						
Charger	Green - flashes					
Charged	Green					

Daily cleaning and maintenance

Clean with a damp cloth and ordinary detergent. Do not use strong acids, bases or alcohol when cleaning. See user manual.





Scan here for user manual, etc. concerning use of the GL5.2.

Accessories

Lifting hanger

Place the sling strap in the hooks. Upon lifting make sure that the strap maintains its position.

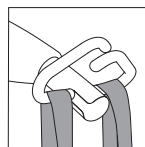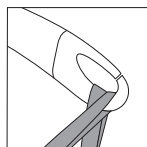

| Time to care |

V. Guldmann A/S

Graham Bells Vej 21-23A, DK-8200 Aarhus N
Tel. +45 8741 3100
info@guldmann.com
www.guldmann.com