



Guldmann™

Guldmann Basic High Sling Skills Check

	Key Element	Satisfactory	Unsatisfactory	Needs Practice	Comment or N/A
1.	Always explain procedure to patient or resident before and during task. When working with others during task, always communicate and verify actions.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
2.	Verbalizes how to inspect the sling for damage, wear and tear, and what criteria indicates when it's time to remove a sling from service.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
3.	Verbalizes and demonstrates where to verify the weight capacity (SWL) of the Guldmann Basic High sling.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
4.	Verbalizes and demonstrates how to select the appropriate size for the patient or resident				
5.	Verbalize understanding that the lowest weight capacity between equipment and sling will determine the safe working load for the task (i.e. if the hoist or hanger or sling or other accessory vary in SWL).	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
6.	Verbalizes primary and secondary functions of the Guldmann Basic High sling.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
7.	Verbalizes understanding that loops on the Guldmann Basic High sling need to match to its opposite side(s) (i.e. both shoulder straps should have the same color loop attached – both grey or both green; both leg straps should have the same color loop). It is acceptable to vary the color loop attached between the shoulders and legs.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	



Guldmann™

Guldmann Basic High Sling Skills Check

	Key Element	Satisfactory	Unsatisfactory	Needs Practice	Comment or N/A
8.	Demonstrates attention to ensure slings loops are securely on the hanger bar's hooks before it is loaded with weight and after patient is lifted/before moving away from surface.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
9.	Identifies the Basic high features to ease sling placement: pocket; center contrast stitching; plastic stays.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
10.	Demonstrates how to remove the Guldmann Basic High sling after transfer, and verbalize how/where to store sling when not in use.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
11.	Verbalize proper cleaning and facility laundering protocol for reusable slings, as applicable, and where clean slings are stored.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
12.	Verbalizes awareness of resource material to answer questions on sling use and functionality, manufacturer's recommendations for cleaning, and Inspection Guidelines / when to remove a Guldmann Basic High sling from inventory.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	