



Guldmann™

Guldmann Trainer Sling Skills Check

	Key Element	Satisfactory	Unsatisfactory	Needs Practice	Comment or N/A
1.	Always explain procedure to patient or resident before and during task. When working with others during task, always communicate and verify actions	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
2.	Verbalizes knowledge of Guldmann Sling Inspection Requirements and laundering guidelines (reusable slings only)	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
3.	Verbalizes and demonstrates where to verify the weight capacity (SWL) of the Guldmann Trainer sling.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
4.	Verbalizes and demonstrates how to select the appropriate size for the patient or resident				
5.	Verbalizes primary and secondary functions of the Guldmann Trainer sling.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
6.	Verbalizes and demonstrates appropriate anatomical area for placement of the Guldmann Trainer sling	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
7.	Verbalizes and demonstrates how to adjust the Trainer sling on the patient / resident for secure and effective fit	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	



Guldmann™

Guldmann Trainer Sling Skills Check

	Key Element	Satisfactory	Unsatisfactory	Needs Practice	Comment or N/A
8.	Demonstrates attention to ensure slings loops are securely on the hanger bar's hooks before it is loaded with weight and after patient is activated.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
9.	Identifies the Trainer sling features to ease sling placement and alignment on patient or resident	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
10.	Verbalize proper cleaning and facility laundering protocol for reusable slings, as applicable, and where clean slings are stored	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
11.	Verbalizes awareness of and where to find resource material on Guldmann Trainer sling's use and functionality and manufacturer's guidelines	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	