



Guldmann GH3 Twin Quiz

Guldmann™

1. A user activates the Guldmann GH3 Twin hoist by:

- A. Moving the hanger and rails
- B. Pulling on the red cord
- C. Pressing an arrow button on the hand control**
- D. All of the above

Answer C

2. When is using a Guldmann GH3 Twin hoist appropriate?

- A. When patient is unable to reposition and roll themselves in bed
- B. When a patient is unable to stand and walk safely
- C. When a patient requires one or more limbs or body parts lifted and supported
- D. When patient needs to be transferred to a chair or bed
- E. All of the above**

Answer E

3. How do you know the hoist requires service attention?

- A. Indicator light glows yellow amber
- B. Indicator light glows green
- C. Hoist will emit a low tone beep
- D. A and C**

Answer D

4. How do you verify the weight capacity of the hoist?

- A. Manufacturer's labels on the hoist**
- B. Refer to the Product Manual
- C. Ask the patient/ resident
- D. Ask a co-worker

Answer: A

5. How do you engage an emergency stop?

- A. Through the hand control function
- B. Call BioMed department
- C. Pull the red cord**
- D. All of the above

Answer: C

Guldmann GH3 – Quiz Questions

6. How do you engage a manual lowering function?

- A. Through the hand control function
- B. Remove the sling and patient from the hanger
- C. Pull the red cord
- D. Pull and hold down the red cord until lowering is complete**

Answer: D

7. How do you reactivate the hoist after emergency stop or manual lowering?

- A. Through the hand control function
- B. Push the Reset button up until slightly recessed and push button on hand control**
- C. Remove the hanger from the lifting strap
- D. All of the above

Answer: B

8. Which item is NOT a responsible caregiver use of the GH3 Twin hoist?

- A. Guarding the hand control when not in use by parking it on the hanger
- B. Guarding the hanger when it is lowered and in proximity to patient/resident or co-workers when it is not stable and loaded with weight
- C. Allowing patient/resident and visitors to use the hoist without permission, instructions, supervision**
- D. Referring to the Product Manual and Manufacturer's literature for guidance

Answer: C

9. How many persons should operate the Guldmann GH3 Twin hoist?

- A. Minimum of 1 person
- B. Indications for 2 or more persons are patient and task dependent
- C. Indications for more than 1 person are determined by the facility and their governing rules and regulations
- D. All of the above**

Answer: D

10. The GH3 Twin hand control has 4 directional arrow buttons - up, down, right, and left. The left and right arrows adjust the twin straps' length and provide a tilt function.

- A. True
- B. False

Answer: A