



Guldmann GH3 Twin Skills Check

Guldmann™

	Key Element	Satisfactory	Unsatisfactory	Needs Practice	Comment or N/A
1.	Always explain procedure to patient or resident before and during task. When working with others during task, always communicate and verify actions	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
2.	Verbalizes how to verify the weight capacity (SWL) of the hoist - 1100 lbs.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
3.	Verbalizes understanding that the lowest weight capacity will determine the safe working load for the task (i.e. if the hanger or sling or other accessory vary in SWL)	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
4.	Demonstrates ability to activate the hoist and understands purpose and function for the hand control arrows which adjust the twin straps / tilt function of the GH3 Twin	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
5.	Demonstrates attention to guard the hand control and the hanger's stability when lowered and in proximity to patient / resident or colleagues before it is loaded with weight.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
6.	Verbalizes understanding of what the indicator light colors signify: green 'awake' and yellow/amber 'no power or no charging'	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
7.	Verbalizes understanding of how the GH3 Twin lift charges	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
8.	Demonstrates ability to perform 'emergency stop' function (quick pull on red cord)	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
9.	Demonstrates ability to perform a 'manual lowering' function (sustained pull downward on red cord)	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
10.	Demonstrates ability to reset the hoist to reactivate its functions	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
11.	Verbalizes awareness of how to request a service check for the hoist	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
12.	Verbalizes awareness of and how to find resources materials for inform questions on operation, care, and service of the hoist	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	